


# N95 Cover Face Mask Pattern

*"Mask for Val" aka PVPA Mask*

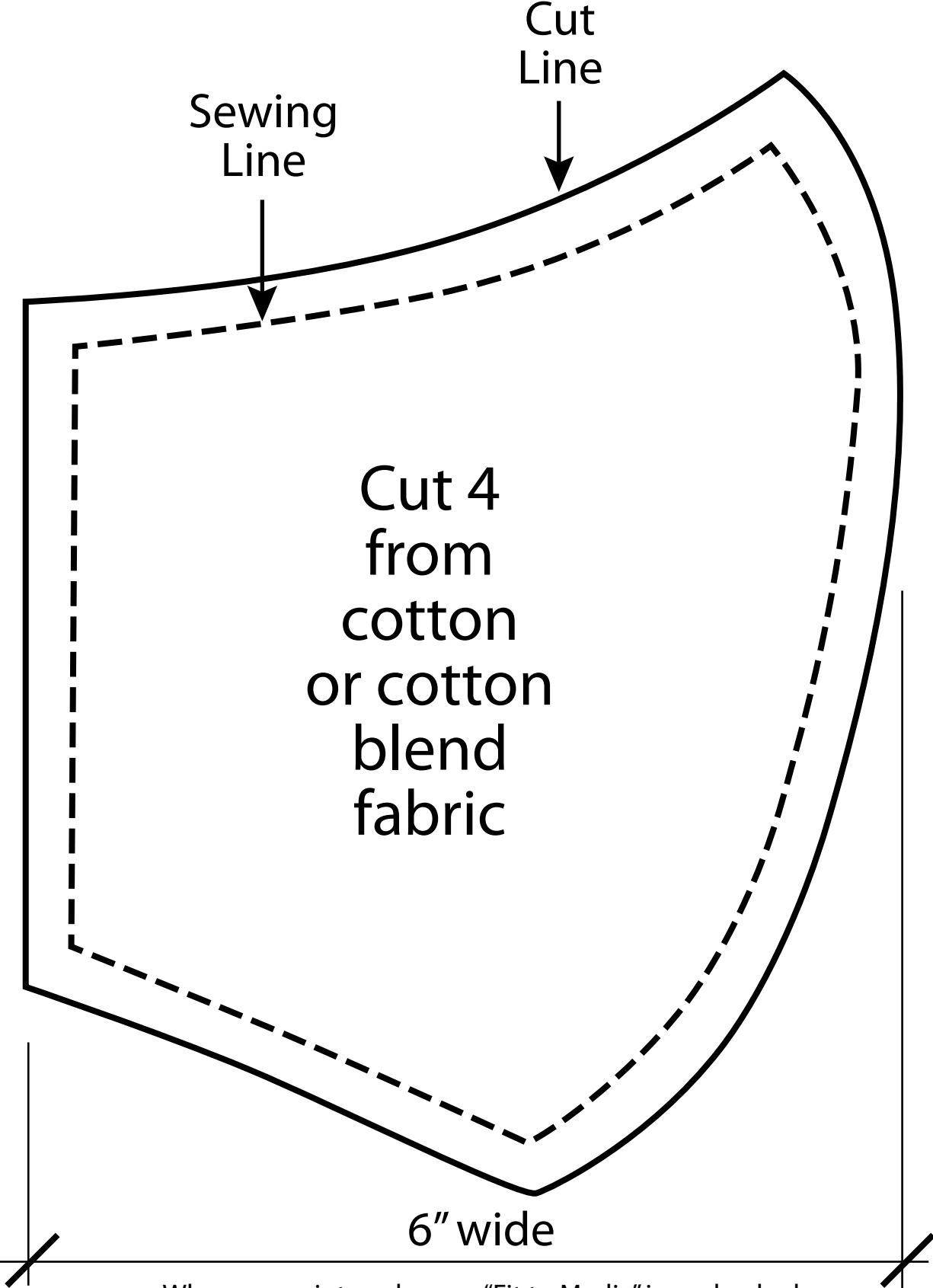

When you print, make sure "Fit to Media" is unchecked so this pattern prints out at 100% scale

# Cover for N95 Mask (PVPA Style)

*Pattern on other side (or page 2) of this flyer.*

## Materials:

Fabric – Cotton or cotton blend.



*Note, photo on right: Elastic substitute, polar fleece*

Inner layer should ideally be a lighter color than the outside. Outer layer can be decorative or of a darker color than the inside. It's important to be able to tell the inside from the outside of the mask easily. Bedsheets are a common fabric and suitable for the sewing this pattern.



*Note light color on inside vs. contrasting decorative fabric which indicates outside of mask.*

Elastic – 1/4” or narrower, if you have it. Try searching “elastic” on eBay

## SEWING INSTRUCTIONS

1. With right sides together, sew the curved center seam. Repeat this step for both the outer and inner fabric.
2. Lay top layer right side uppermost on a surface. Place an 11” elastic to top corners with ends slightly extended from the sides. Place inner layer with **WRONG** side uppermost over top layer. Pin in place.
3. Sew down the sides, securing the elastic. Sew the top. Sew the middle of the bottom, catching in the center seams.
4. Sew from the bottom center to one side. This will leave a 2” opening on the bottom
5. Turn mask inside out and sew opening closed and edge stitch around entire mask.
6. Fold a pleat on each side and insert 9-1/2” elastic. Sew to secure.